
**Owner's Manual  
Manual del Usuario  
Guide de L'utilisateur  
Gebrauchsanweisung**

**Envoy™**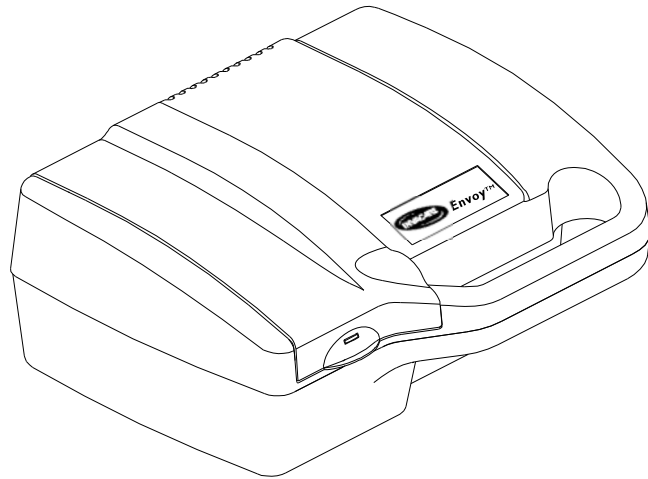

**AEROSOL COMPRESSOR  
MODEL IRC1193, IRC1194A, AND IRC1194B**

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**AEROSOL KOMPRESSOR  
MODELLE IRC1193, IRC1194A UND IRC1194B**

**SPECIAL NOTES**

DANGER/WARNING/CAUTION notices as used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

**NOTICE**

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.  
THIS AEROSOL COMPRESSOR IS INTENDED FOR HOUSEHOLD USE.

**WARNING**

DO NOT install, maintain or operate this equipment without first reading and understanding this manual. If you are unable to understand the DANGERS / WARNINGS / CAUTIONS and INSTRUCTIONS, contact your Home Healthcare Dealer before attempting to use this equipment - otherwise injury or damage may result.

**WARNING**

DO NOT dispose of the Envoy aerosol compressor in the household waste at the end of its useful life. Proper disposal at an appropriate collection site for electronic devices in your local area is strongly recommended and may be required.

**SAVE THESE INSTRUCTIONS**

TABLE OF CONTENTS

T  
A  
B  
L  
E  
O  
F  
C  
O  
N  
T  
E  
N  
T  
S

SPECIAL NOTES ..... 2

IMPORTANT SAFEGUARDS ..... 4

SPECIFICATIONS AND SUPPLEMENTAL INFORMATION ..... 5

FEATURES ..... 6

HANDLING PROCEDURES ..... 7

    SHIPPING AND TRANSPORTATION INSTRUCTIONS ..... 7

    UNPACKING ..... 7

    INSPECTION ..... 7

    STORAGE ..... 7

TROUBLESHOOTING GUIDE ..... 7

OPERATION ..... 8

MAINTENANCE ..... 10

    CLEANING THE OUTER CASE ..... 10

    CLEANING THE NEBULIZER ..... 10

    CLEANING OR REPLACING THE FOAM INTAKE FILTER ..... 10

ACCESSORIES AND LIMITED WARRANTY ..... 11

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**IMPORTANT SAFEGUARDS**

**READ ALL INSTRUCTIONS BEFORE USING**

“Caution: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.”

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

**DANGER  
TO REDUCE THE RISK OF ELECTROCUTION**

ALWAYS unplug this product IMMEDIATELY after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water. UNPLUG IMMEDIATELY.

DO NOT place or drop into water or other liquid.

**WARNING  
TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS**

This product should NEVER be left unattended when plugged in.

Close supervision is necessary when this product is used by, on, or near CHILDREN AND PHYSICALLY CHALLENGED INDIVIDUALS.

Use this product only for its intended use as described in this manual. Use this product only under physician’s direction. DO NOT use attachments not recommended by the manufacturer.

**WARNING**

NEVER operate this product with a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water; call Qualified Service Personnel for examination and repair.

Keep the cord away from HEATED or HOT surfaces.

NEVER drop or insert any object into any opening.

NEVER use while sleeping or drowsy.

NEVER block the air vent openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free from lint, hair and the like. Blocked openings may cause the unit to shut down. Contact dealer immediately.

DO NOT use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered in a closed environment such as an oxygen tent.

This unit is oil-less. DO NOT lubricate.

**CAUTION**

Risk of electric shock. DO NOT disassemble. Refer servicing to qualified service personnel.

**SPECIFICATIONS AND SUPPLEMENTAL INFORMATION**

**STANDARD SPECIFICATIONS:**

IRC1193 (2-wire Europlug), IRC1194A (3-wire Australian plug) and IRC1194B (3-wire UK plug)

 :  :  <b>(IRC1193 ONLY):</b>  <b>(IRC1194A and IRC1194B ONLY):</b>  : Electrical Requirements IRC1193 and IRC1194B: IRC1194A: Running Current: Sound Level: Ambient Temperature Range: Power Wattage: Thermal Protector: Foam Intake Filter: Safety Systems: Width: Height: Depth: Weight: Shipping Weight: Operating Pressure: Operating Flow Rate:	Alternating current. Type BF degree of protection against electrical shock. Class II type protection against electrical shock. Protective Earth (Ground) Attention - Consider Accompanying Documents 220/230 V~ 50Hz, Single Phase 220/230/240 V~ 50Hz, Single Phase 0.6 A @ 0.70 Bar (10 psig) @ 230 volts (approx.) 55 dBA 10°C - 40°C (50°F - 104°F) 65 watts @ 0.70 Bar (10 psig) UL 547 Class B Scot Industrial Foam, open cell 100 ppi, polyurethane foam. High temperature compressor shutdown. 27.9 cm (11-inches) 11.2 cm (4.4-inches) 21.6 cm (8.5-inches) 2.1 kg (4.6 lbs.) 2.8 kg (6.2 lbs.) 0.70-0.83 Bar, typical (10-12 psig, typical) 5 lpm, typical
---	--

**ELECTRICAL - Compressor:**

If the unit shuts down frequently, you may have a low voltage situation. Low voltage can also be suspected when:

1. The motor does not get up to full power or speed.
2. Home fuses or circuit breakers activate when starting compressor.
3. Lights dim or remain dim when compressor is started.
4. Other motor operated appliances fail to operate properly or too many motor operated appliances are on same circuit.

The motor of this compressor has a thermal overload protector. If the motor should overheat, the overload protector will shut off the motor. If the unit shuts down and cannot be restarted, the motor must be replaced. Call your dealer immediately.

**Extension Cords:**

Make certain extension cord is in good condition. Make certain extension cord wire is NOT LESS than 0.75 mm<sup>2</sup> (18 gauge) for 7.6m (25 ft.), 1.00 mm<sup>2</sup> (16 gauge) for 15m (50 ft.) and 1.50 mm<sup>2</sup> (14 gauge) for 30m (100 ft.).

**MAINTENANCE**

Only authorized or factory-trained personnel should perform preventative maintenance.

FEATURES

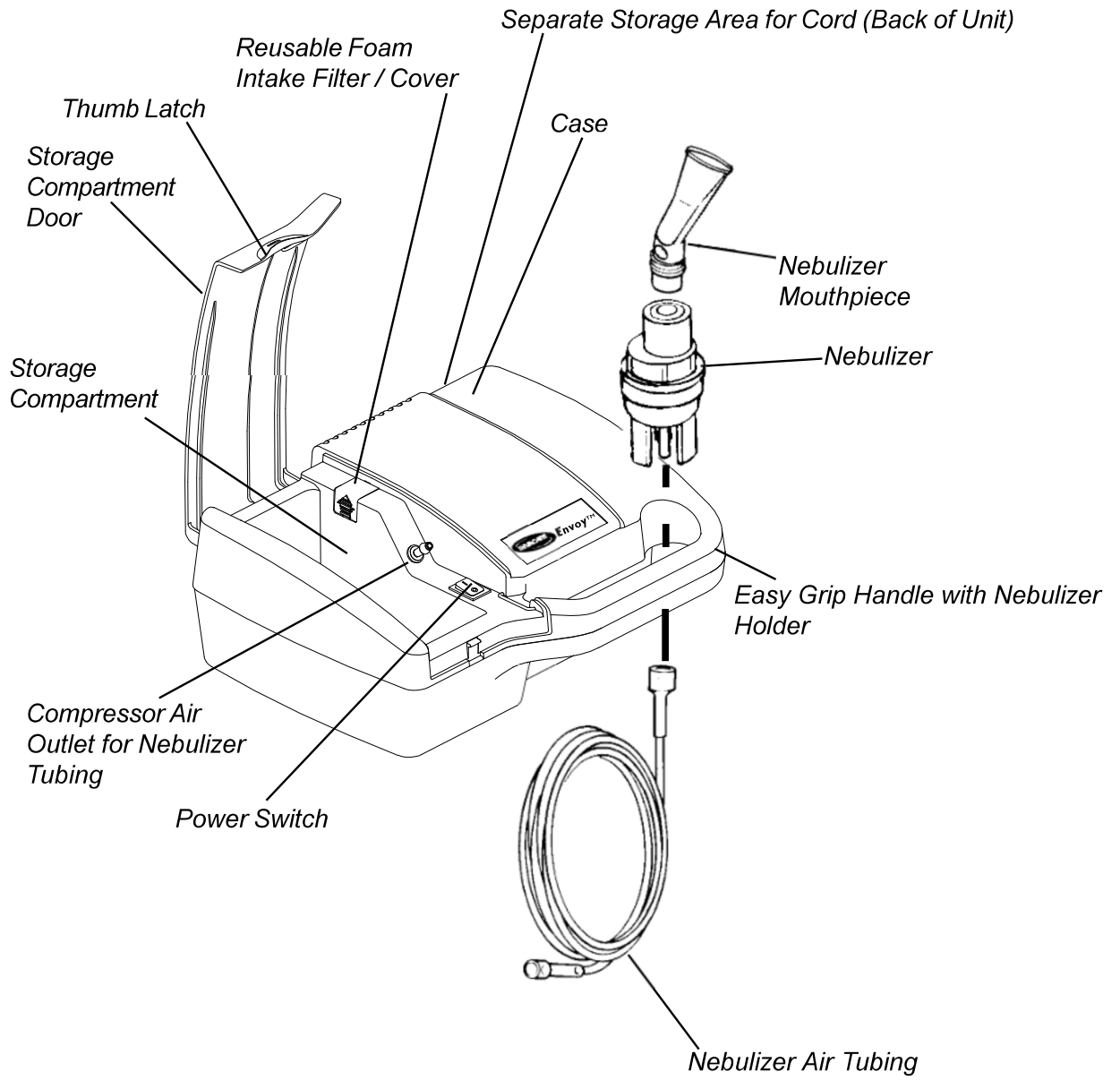
FIGURE 1 - FEATURES

**HANDLING PROCEDURES****SHIPPING AND TRANSPORTATION INSTRUCTIONS**

The shipping container has been designed to assure protection of this device.

If this device is to be reshipped, it should be packed in the same carton. Extra cartons are available from Invacare.

**UNPACKING**

1. Check for any obvious damage to the carton or its contents. If damage is evident, do NOT use. Notify Carrier/ Invacare or your local Invacare Dealer.
2. Remove all the loose packing from the carton.
3. Carefully remove all the components from the carton.

*NOTE: Retain carton and packing materials for use in shipping or storing the unit until use is required.*

**INSPECTION**

1. Examine exterior for nicks, dents, scratches or other damage. Inspect all components.

**STORAGE**

1. Store the repackaged device in a dry area.
2. DO NOT place other objects on top of the repackaged device.

**TROUBLESHOOTING GUIDE**

SYMPTOM	POSSIBLE CAUSE	SOLUTION
Home fuses or circuit breakers activate when starting compressor.	Too many appliances on the same circuit.	Remove other appliances operating on the same circuit.
Lights dim (or remain dim) when compressor starts.	Too many appliances on the same circuit.	Remove other appliances operating on the same circuit.
Compressor temporarily shuts down or stalls.	Too many appliances on the same circuit. Remove the nebulizer tubing from the air outlet fitting.	Remove other appliances operating on the same circuit. Connect the nebulizer tubing when the compressor is at full speed.
Compressor does NOT get to full speed or power.	Too many appliances on the same circuit. Remove the nebulizer tubing from the air outlet fitting.	Remove other appliances operating on the same circuit. Connect the nebulizer tubing when the compressor is at full speed.
Compressor shuts down and can't be restarted.	Internal repairs needed.	Contact your dealer.

**OPERATION**

Your disposable nebulizer package includes a nebulizer mouthpiece, nebulizer, and air tubing. These items are sold as a package only.

*NOTE: Before initial operation, nebulizer should be cleaned. Follow the recommended Cleaning Instructions in this Owner's Manual.*

**WARNING**

DO NOT run unit without filter.

Avoid operating this product in a dusty environment which could cause premature compressor wear.

**SETUP**

1. Place the device on a table so that the controls can be reached when seated.
2. Open the storage compartment door by lifting up the latch with your thumb and pulling out with your fingers.
3. Remove nebulizer, nebulizer mouthpiece, and air tubing from the storage compartment.
4. Secure one end of the tubing to the compressor air outlet (**FIGURE 1**).
5. Connect the other end to the nebulizer.

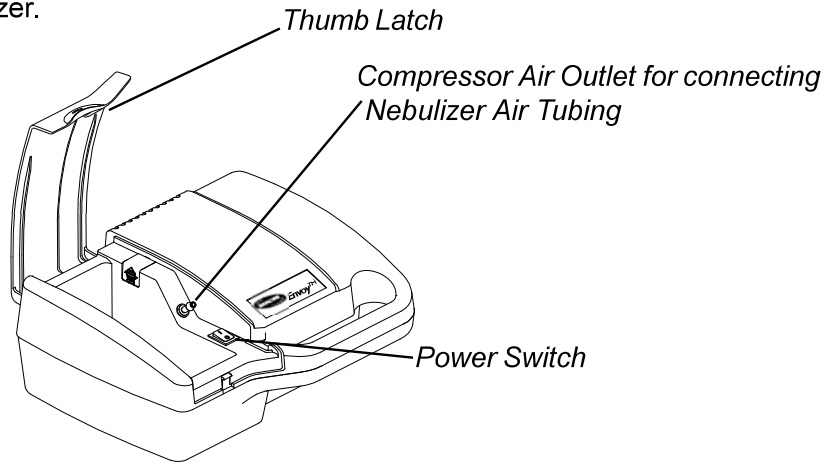
FIGURE 1 - LOCATION OF COMPRESSOR AIR OUTLET

**WARNING**

In the case of high humidity, condensation may form inside the internal tubing. Run the unit for two (2) minutes before connecting the nebulizer to the tubing.

6. Add liquid medication to the nebulizer according to your physician's prescription (**FIGURE 2**).

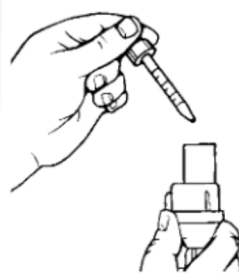
FIGURE 2 - ADDING LIQUID MEDICATION TO THE NEBULIZER

**WARNING**

Before plugging in the line cord, make sure that the power switch is in the OFF (O) position.

- Remove the line cord from the storage bin in the rear of the unit (**FIGURE 3**).

**WARNING**

Inspect the line cord. If the line cord is damaged, DO NOT turn on the unit. Call your Dealer or Qualified Invacare Representative.

- Plug unit into a wall outlet with appropriate voltage and plug type.

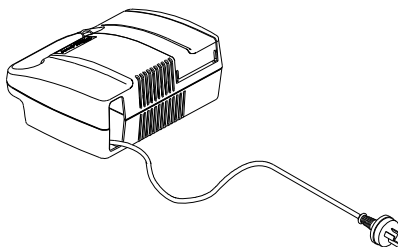
FIGURE 3 - LINE CORD LOCATION

- Insert the nebulizer mouthpiece into the top of the nebulizer.

*NOTE: Before attempting to start a cold unit, let the unit reach room temperature in a heated area.*

- Press the power switch ON (I) to start the compressor.

**WARNING**

DO NOT block the Compressor Air Outlet while the compressor is running. Blocking the air outlet may damage the compressor.

- Place mouthpiece between your teeth and close your mouth (**FIGURE 4**).
- As aerosol begins to flow, inhale deeply, slowly, and hold breath for 10 seconds. Then exhale slowly through the mouthpiece.

*NOTE: If you need to rest during a treatment, place the nebulizer in the round holder located within the handle opening. (Refer to Features Section in this manual.)*

*NOTE: Unit may be operated with the storage compartment door closed. Route nebulizer air tubing through side opening in storage compartment door.*

*NOTE: Before storing supplies between treatments, clean and dry nebulizer. (Refer to Maintenance Section in this manual.) Tighten caps on medication jars.*

FIGURE 4 - USING THE NEBULIZER MOUTHPIECE

MAINTENANCE

WARNING

Electrical shock hazard. DO NOT remove outer case of this unit. All disassembly and maintenance of this unit MUST BE done by a qualified Invacare Dealer.

CLEANING THE OUTER CASE

WARNING

UNPLUG unit before cleaning. DO NOT submerge in water to clean. Clean outer case with damp cloth.

CLEANING THE NEBULIZER

*NOTE: With proper maintenance your nebulizer should last 30 days. Since the nebulizer is disposable, it is recommended that an extra nebulizer be kept at all times.*

*NOTE: Clean the Nebulizer after every use and at least once a day.*

*NOTE: Your physician and/or a Invacare Dealer may specify certain cleaning procedures. If so, follow their recommendations. If not, clean according to the following instructions:*

1. Remove the nebulizer and mouthpiece from the tubing.
2. Disassemble nebulizer chamber by turning counterclockwise and separating.
3. **Fill two (2) plastic containers or bowls:**
  - a. **Washing:** Hot water/detergent solution.
  - b. **Soaking:** Hot water/vinegar solution (one [1] part vinegar to three [3] parts water).
4. Thoroughly clean the nebulizer and mouthpiece in hot water/detergent solution.
5. Remove from the water/detergent solution and rinse with clear hot tap water.
6. Soak in hot water/vinegar solution for 30 minutes.
7. Rinse with hot tap water again and air dry thoroughly.

*NOTE: If using medical disinfectant cleaners, follow manufacturer's instructions carefully.*

8. Keep the outer surface of the tubing dust-free by wiping regularly.

*NOTE: The nebulizer air tubing does not have to be washed internally because only filtered air passes through the tubing.*

CLEANING OR REPLACING THE FOAM INTAKE FILTER

*NOTE: Clean the filter once a month or sooner if filter discolors.*

1. Unplug unit.
2. Open cover of storage compartment to gain access to filter.
3. Remove filter and clean with soap and water. Let dry thoroughly before use.
4. Replace filter if it becomes clogged, torn or has a worn appearance.

*NOTE: Regular filter cleaning/replacing is necessary to help ensure proper compressor performance.*

WARNING

DO NOT run unit without filter.

## ACCESSORIES

Intake Filter

Filter Cover

Owner's Manual

\*MS2200 Disposable Sidestream Nebulizer (10 ml Capacity), w/Mouthpiece and 7 ft. Tubing.

MS2400 Durable Sidestream Nebulizer (10 ml Capacity), w/Mouthpiece and 7 ft. Tubing. One (1) Year Limited Warranty.

NOTE: See *Invacare Dealers for additional Nebulizers.*

*\*Not available in all areas.*

### LIMITED WARRANTY

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK / Invacare Service Representative warrants the compressor of the Invacare Envoy Aerosol Compressor to be free from defects in materials and workmanship for a period of three (3) years from date of purchase. The nebulizer chamber, nebulizer air tubing, and any other accessories are excluded from cover under this warranty. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option ; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

Limitations and Exclusions : The foregoing warranty does not cover normal wear and tear and shall not apply to products subjected to negligence, accident, improper operation, maintenance or storage, commercial or institutional use, products modified without Invacare's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) or to products damaged by reason of repairs made to any component without the specific consent of Invacare UK / Invacare Service Representative.

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© 2001 Invacare Corporation
Invacare is a registered trademark of
Envoy is a trademark of Invacare Corporation.
Invacare Corporation [www.invacare.com](http://www.invacare.com)
Invacare Corporation.